


# Technical Data sheet

Baltic Masks, BM-920 Disposable MEDICAL Face Mask ( V2.0 )

A 3ply face mask is a loose-fitting, disposable device that creates a physical barrier between the mouth and nose of a 3ply face mask is a loose- fitting, disposable device that creates a physical barrier between the mouth and nose of the wearer and potential contaminants in the immediate environment. Contains no glass fiber, latex, nickel. Hypoallergenic.

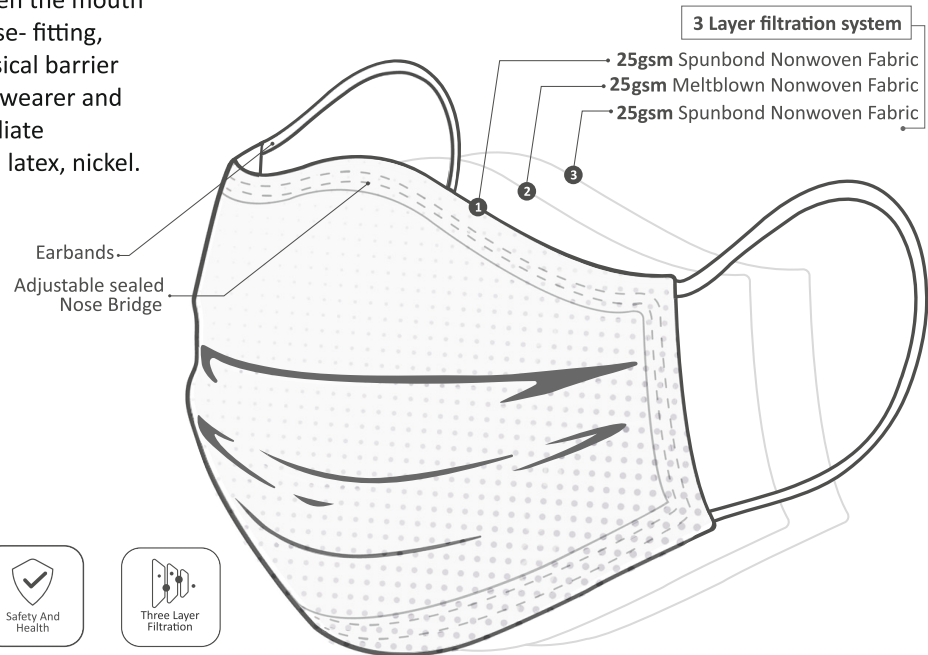

**MANUFACTURER:**

**Changzhou Shuangma Medical Devices Co., Ltd.**  
San He Kou Development Zone, Zhenglu, Tianning, Changzhou, 213115, Jiangsu, China

**MEDICAL DEVICE:**

**MEDICAL DISPOSABLE NON STERILE FACE MASK**

**MODEL:**

**BM-920**

**CLASSIFICATION:**

**CLASS I**

**QUALITY ASSURANCE AND CONFORMITY:**

Directive: **Medical Device Directive 93/42/EEC**  
Standard: **EN 14683:2019+AC:2019**  
Type: **IIR**

**MATERIALS:**

Outer Material **1st layer - 25gsm Spunbond Non-Wowen**  
Filter Layer **2nd layer - 25gsm Melt-Blown fabric**  
Inner Material **3rd layer - 25gsm Spunbond Non-Wowen**  
Nose-piece **Plastic single wire thread 3mm**  
Earloops **3mm Polyester and Spandex**

**SIZE:**

Mask body **175x95mm (+/- 5mm)**

**SHELF LIFE:**

**2 years after production date**

**STORAGE:**

**Storage under cool, clean, and dry conditions. Avoid excessive heat ( -20°C to +40°C and at less than 80% relative humidity )**


14/09/2020

# Technical Data sheet

Baltic Masks, BM-920 Disposable MEDICAL Face Mask ( V2.0 )

## Packaging

### 50 pcs in a carton box

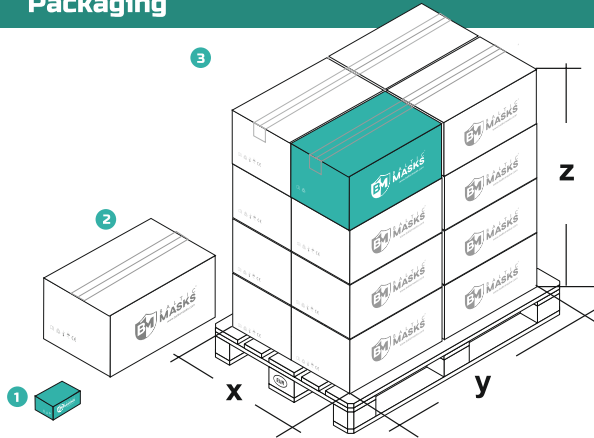

- 1 **Size:** 194mm (y) x 97mm (x) x 80mm (z)  
**Weight:** 210g  
**Quantity:** 1pack ( 50pcs.)

---

- 2 **Size:** 520mm (y) x 390mm (x) x 350mm (z)  
**Weight:** 8.8kg  
**Quantity:** 1/40pack ( 2000pcs.)

---

- 3 **Size:** 1040mm (y) x 780mm (x) x 1750mm (z)  
**Weight:** 135kg  
**Quantity:** 20/40pack ( 40.000pcs.)

## Container loading

**Container size:** 20'DC

**Loading:** Carton box, free loading

**Quantity:** 400 carton / 800 000pcs.

**Container size:** 40'DC


**Loading:** Carton box, free loading

**Quantity:** 800carton / 1 600 000pcs.

**Container size:** 40'HC

**Loading:** Carton box, free loading

**Quantity:** 970carton / 1 940 000pcs.

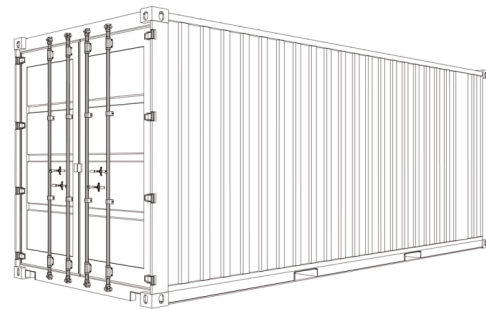

**Storage Conditions and Shelf Life** The filter masks have to be stored in their packets to retain their properties. Shelf life of unopened product is 2 years from date of manufacture when stored within temperature range of -20°C to +40°C and at less than 80% relative humidity.

# Technical Data sheet

Baltic Masks, BM-920 Disposable MEDICAL Face Mask ( V2.0 )

## Usage instruction



1. Before opening and touching the mask, wash your hands thoroughly with soap and water or alcohol/based hand sanitizer (>60% alcohol content).
2. Open the mask with both hands to the two sides.
3. Hold the mask by the ear loops. Place a loop around each ear and be sure the nose clip is upwarded
4. Press the nose clip to fit the shape of the nose
5. Pull the bottom of the mask over your mouth and chin

## Precautions

1. This product is not suitable for protection against harmful gases, steam and oily aerosols.
2. The product does not generate oxygen and is not suitable for use in anoxic environments, underwater operations, during evacuation or firefighting environments.
3. It is not recommended to use this protective mask if you do not feel a good adhesion between the face and the edge of the mask during use.
4. If found during use that the mask is damaged or the breathing resistance is too large, replace with a new one.
5. This product is for one-time use, masks should not be reused.